FR

FILT'RAY - FILTRES TERMINAUX ANTI-TOUS GERMES STERILES

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1 DESCRIPTION

EN

The medical devices covered by this notice are anti-all germ filters for taps and/or showers. They are commonly called "**FILT'RAY tap filters**" for microfiltration cartridges and "**Screw-on shower filters**" for shower heads fitted with a microfiltration cartridge.

They are installed at the end of the drinking water network (tap or shower) and serve as **physical barriers in the event of a bacterial alert** (*Legionella pneumophila*, *Pseudomonas aeruginosa*, and other Gram-negative bacilli, non-tuberculous Mycobacteria, fungi, protozoa and other particles from the water distribution network – absolute filtration grade of 0.2 µm) for an **indicative duration of 1, 2, 3 or 4 months**. The protection of immunocompromised patients and vulnerable people is ensured thanks to the microfiltration cartridge. These medical devices are for single use only.

They are intended to be installed in hospitals or medical care facilities (retirement homes, specialised centres), as well as in collective facilities such as leisure or sports centres, hotels, etc.

2 REFERENCES

- FILT'RAY screw-on shower filter: AT20161, AT20261, AT20361, AT20461.
- FILT'RAY Tap Straight Jet: AT20151, AT20251, AT20351, AT20451.
- FILT'RAY Tap Shower head Jet: AT20150, AT20250, AT20350, AT20450.

To guarantee optimal performance of the FILT'RAY filters, please comply with the installation protocol (see 3) and the precautions for use (see 7).

The list is available on the website: www.aqua-tools.com, or you can get it from our teams: tel.: + 33 (0)1 39 75 02 20 - e-mail: sav@aquatools.fr.

3 INSTALLATION

Warnings:

Wash and disinfect your hands before installing the filter to limit the risk of contamination or back contamination.

Check that the single packaging bag is not punctured or torn and that the filter is not damaged.

If necessary, dispose of the filter as its safety and performance are no longer guaranteed.

Check that the expiry date has not passed. If necessary, discard the filter because its safety and performance are no longer guaranteed.

Preliminary preparation of the taps:

- The screw-on shower filter: it is screwed quickly clockwise and until the seal is crushed on all shower hoses complying with standard NF EN 1113.

The length of the hose must be adapted to avoid any contact between the shower filter and the ground. The gasket provided must be installed to ensure sealing and prevent possible leaks. Each time the shower head is changed, it should be replaced.

- The tap filter: it must be fitted using the quick connectors recommended by AQUATOOLS. The quick connectors allow to connect and disconnect the filter to the tap easily.

Visit the website or contact us for a list of available fittings.

Laying:

1. Open the tap to purge before installation to avoid early clogging.
2. Close or cut off the water supply to an electronic tap.
3. In the case of first installation:
 - Tap filter: place an adapter fitting.
 - Screw-on shower filter: remove the original shower head.
4. If replacing a worn filter: remove the filter.
5. **Wash and disinfect your hands.**
6. Partially open the unit packaging.
7. While keeping the filter in the bag to prevent contamination of the water outlet, screw the spray filter to the end of the hose or clip the cartridge filter to the fitting adapter.
8. Partially open the unit packaging.
9. Turn the water on for a few seconds to check that the filter and fittings remain perfectly sealed.
10. In case of leakage, repeat the procedure. Do not re-use a filter that has been connected.
11. Note the commissioning and the theoretical decommissioning date (commissioning date + theoretical duration of use) on the filter and on the traceability label.

Removal:

1. Close the water supply.
2. Release the used shower head filter by unscrewing it from the end of the hose or unclip the shower head filter or tap filter from the adapter fitting.
3. Dispose of the filter as normal/household waste. Never re-use a filter that has been connected. As the filter cannot be dismantled, the manufacturer cannot guarantee that the fibres are always in good condition.

Installation and use in a dirty or uncontrolled environment can be a source of back contamination. Discard and replace any filter that appears visually soiled or contaminated.

4

CLEANING

The outer surfaces of the filter (not including the water outlet) can be cleaned with a single-use disinfectant wipe, following the cleaning protocols commonly implemented in healthcare establishments (wear protective equipment, specific protocol, etc.).

5

TECHNICAL PERFORMANCE AND EXPOSURE TO CHEMICAL AND THERMAL TREATMENTS

Conditions of use:

The performance of FILT'RAY filters is guaranteed when the pressure and temperature of the water in the network are respectively between 1 and 5 bar/5 °C and 42 °C. The measured outlet water flow rates are as follows:

FILT'RAY filter					
	Water pressure (bar)				
	1	2	3	4	5
Flow rate (L/min) - Tap 1M	2.7	4	5.1	6	7
Flow rate (L/min) - Tap 2M	3	4.4	5.5	6.5	7.4
Flow rate (L/min) - Tap 3M	4.6	6.6	7.6	8.6	9.4
Flow rate (L/min) - Tap 4M	4.6	6.5	7.6	8.5	9.4
Flow rate (L/min) - Shower head 1M	6	8.8	11.2	12.8	13
Flow rate (L/min) - Shower head 2M	6	8.8	11.2	12.8	13
Flow rate (L/min) - Shower head 3M	6.2	9.2	11.2	12.4	13.2
Flow rate (L/min) - Shower head 4M	6.5	9.6	11.8	13.2	14.1

Exposure to chemical and thermal treatments:

The filter can tolerate the following treatments:

- Heat treatment of 70 °C for 30 minutes (maximum cumulative).
- Chemical treatment: active chlorine.
 - Filter 1M: 100 ppm for 1 hour.
 - Filter 2M: 100 ppm for 2 hours.
 - Filter 3M: 100 ppm for 3 hours.
 - Filter 4M: 100 ppm for 4 hours.

Replace the filter if the chemical or thermal treatment exceeds these thresholds.

Clogging tolerance:

If the water in the drinking water network has an unusual rate of particles or microorganisms in suspension, it is possible to observe a decrease in the output flow rate before the theoretical decommissioning date (date of entry into service + theoretical duration of use).

When the flow rate becomes insufficient, replace the filter.

Warning: Clogging may occur before the theoretical discharge date, but it does not directly affect the patient.

Tip: In the case of early clogging, pre-filtration at different levels of the system is recommended to stop suspended solids and/ or colloids.

6

ELIMINATION

Used filters do not pose a risk of infection. Like all bimaterial packages, they are classified as waste similar to household waste. The instructions and the cardboard box are recyclable (paper, cardboard).

7

PRECAUTIONS FOR USE

- Wash and disinfect your hands before touching or cleaning a filter.
- Purge the tap before installing the filter to avoid early clogging.
- Do not use the filter if the packaging and/or the filter are damaged.
- Avoid collisions or bumps with the filter. Replace if damaged.
- Discard and replace any filter that appears visually soiled or contaminated.
- Do not disconnect the filter while in use.
- The water outlet must not come into contact with skin or hair. In the case of contact, there is a risk of back contamination. It is advisable to replace the filter.
- If there is a decrease in water flow, it is advisable to replace the filter.
- Once disconnected, the filter should be discarded (single use).
- The microfiltration cartridge must not be adapted, modified or opened.
- The filter must not be used for the production of water for pharmaceutical use.
- The filter does not change the potability of the water, and it does not modify its concentration of salts, metals, scale, chemicals, pesticides or dyes.
- Shower filters must always be repositioned on the support after use.

8

UNDESIRABLE SIDE EFFECTS

The studies carried out have not demonstrated any adverse effects on the health of patients.

9

PRODUCT COMPLIANCE

FILT'RAY filters:

- are sterilised by gamma irradiation in accordance with NF EN ISO 11137 standard. Do not resterilise.
- are class Is medical devices compliant with European Regulation EU 2017/745. CE marking: year 2022.
- have international drinking water compliance validations (ACS - France, KTW - Germany, WRAS - United Kingdom).

10

MANUFACTURER DETAILS

FILT'RAY filters are manufactured by:

Registered Office: **AQUATOOLS** | 11, rue Lobineau | 75006 PARIS | France

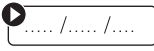
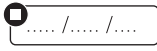
Main establishment: **AQUATOOLS** | 18, rue du Maréchal Foch | 80130 FRIVILLE | France

1. For more information, visit our website: www.aqua-tools.com.
or contact us: **tel.:** + 33 (0)1 39 75 02 20 - **E-mail:** sav@aquatools.fr.
2. The "Scientific and Technical Validation Guideline" relating to filters is available on request.
3. The paper version of the notices remains available on request.
4. Any serious incident must be notified to the manufacturer and to the competent authority of the Member State where the user and/or the patient resides.

Reference:

A sterilising grade filter is defined in the "American Standard Test Method ASTM F838-20 and FDA - Guidance for Industry - Sterile Drug Products Produced by Aseptic Processing - Current Good Manufacturing Practice

Product label - meaning of the pictograms

		
Catalogue reference	Batch code	Expiration date
		
Device sterilised by irradiation	Do not resterilise	Do not use if the packaging is damaged.
		
Manufacturer's address	Meets general safety and performance requirements	Consult the precautions for use
		
Sterile barrier system only.	Medical Device	Unique Device Identifier
		
Do not reuse	Commissioning date	Decommissioning date
		
Manufacturing date	Use-by date after commissioning 1M: 1 month - 2M: 2 months - 3M: 3 months - 4M: 4 months	

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